Product Description




FORGET YOU'RE EVEN ON YOUR PERIOD.

Flex Disc is:

- Made with body-safe materials
- Designed to prévent irritation
- Easy to insert and remove
- About the same size as a tampon when pinched for insertion



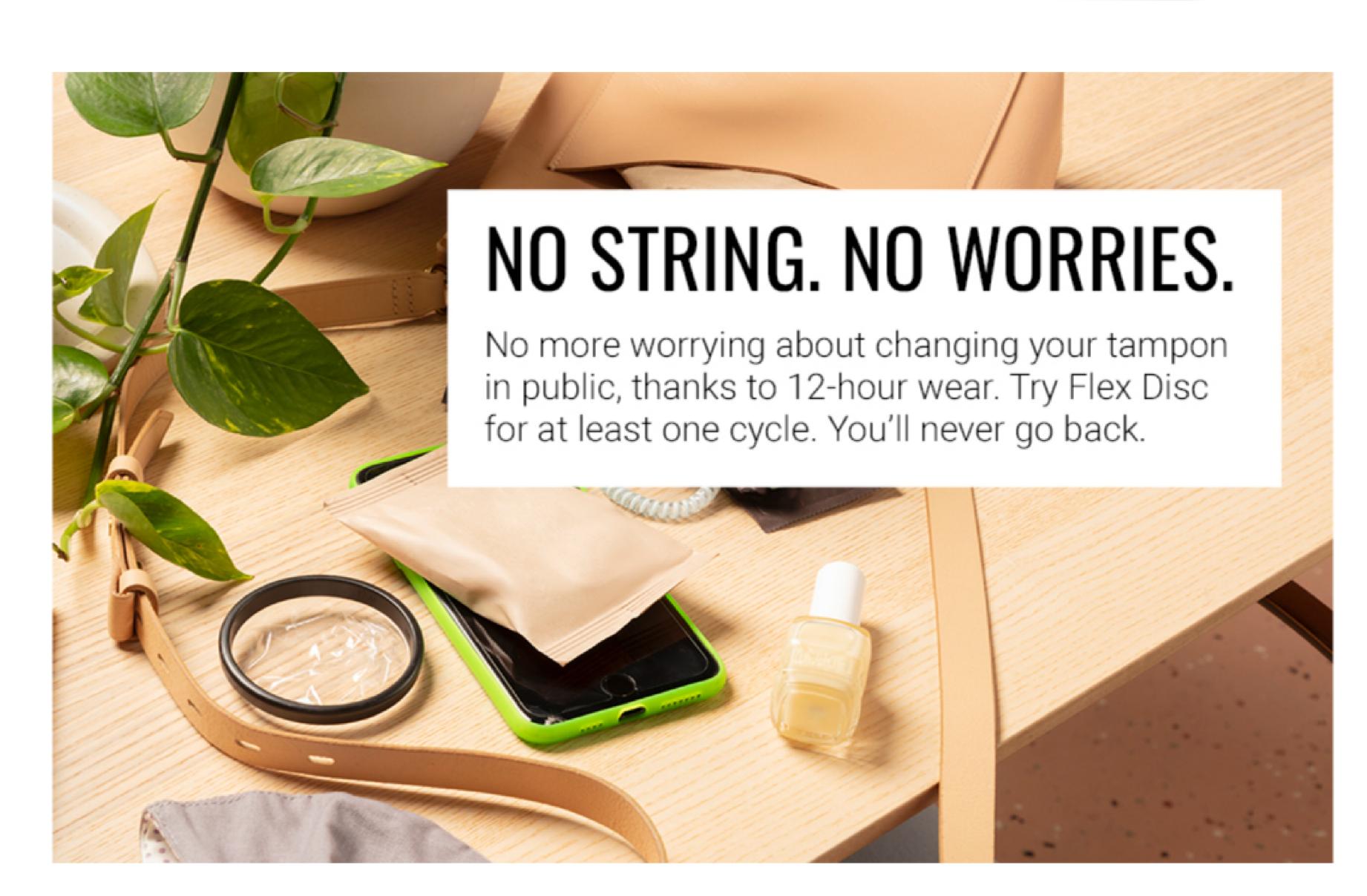




DITCH YOUR TAMPONS IN 3 EASY STEPS

STEP 1: Wash your hands and pinch your disc in half.

STEP 2: Get in position (try sitting, squatting, or standing with one leg up).

STEP 3: Insert the pinched disc down and back. Then, tuck the front rim up behind your pubic bone.

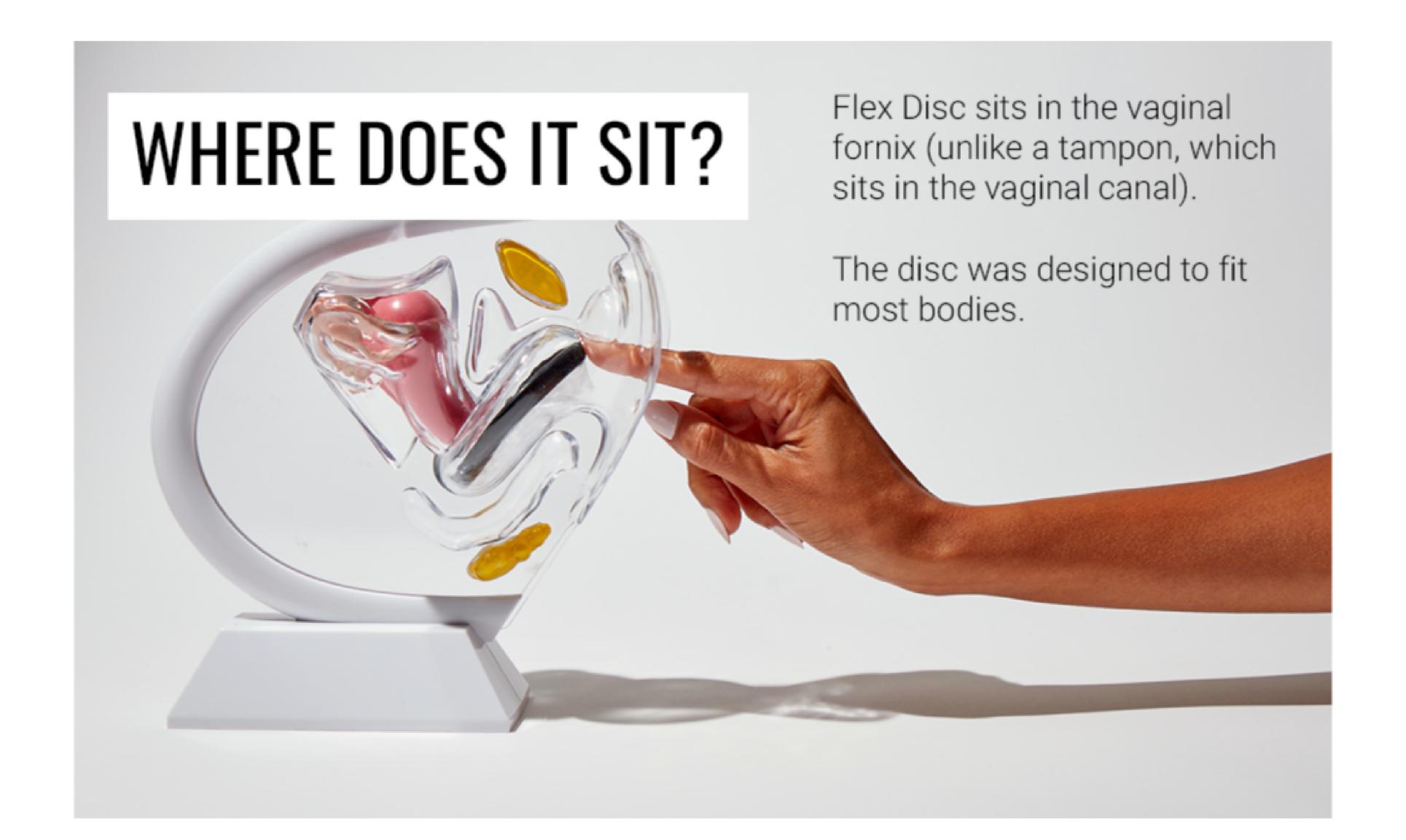

"This product can help you enjoy the glories of sex on your period without any clean up."

GLAMOUR

NYLON

"Discs have several benefits over tampons and pads, including the need to change them less often."

"The Flex team is focusing on helping menstruating people live more freely."

.Mic

"Flex's noble mission is to make it possible for women everywhere to have sex every day of the month."

COSMOPOLITAN









	Flex Disc	Flex Reusable Disc	Flex Cup Size 01 (Slim Fit)	Flex Cup Size 02 (Full Fit)
Capacity	30 mL (3+ super tampons)	70 mL (6+ super tampons)	22 mL (2+ super tampons)	30 mL (3+ super tampons)
Diameter	69 mm	69 mm	43 mm	46 mm
Material	Medical-grade polymers	Medical-grade silicone	Medical-grade silicone	Medical-grade silicone
Placement	Vaginal fornix	Vaginal fornix	Vaginal canal	Vaginal canal
Period product type	Period disc	Period disc	Period cup	Period cup
Cramp-friendly	✓	✓		
Not made with BPA, phthalates, or natural rubber latex	✓	✓	✓	✓
Reusable		✓	✓	✓
Single-use	✓			
Beginner-friendly	✓	✓	✓	✓